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1. Applying Brace to Arm

Hold the brace up to the arm with the hinge pivot center over the elbow joint .

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2. Adjusting Length

To adjust length, press the button on the distal cuffs and slide telescoping bars until they match the length of the upper arm and forearm.

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3. Bending the Cuffs

Bending the cuffs to securely wrap around the circumference of the arm. Start with the cuffs proximal to the elbow.

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4. Secure Straps

After length and cuff circumference are determined, secure and fasten all straps, starting with the two straps closest to the elbow. If the straps are too long, trim them as necessary.

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5. Range of motion adjustments

To adjust range of motion, press the flexion or extension button and rotate until the desired degree of flexion or extension is achieved through the corresponding windows. The "0" lock setting are used to lock the brace in full extension .

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6. Enforced Compliance

Loop and secure the enclosed lock ties through the hole in the flexion and extension buttons to limit range of motion adjustment .

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7. Side Bar Adjustment

Side bars may be bent to fit individual anatomies .

Note side bars must be bent with final length adjustment completed and bent with bending irons .

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Neck Strap Application

1. Attach the D-ring to the strap closest to the wrist.
2. Place the neck strap over the opposite shoulder of the injured arm and across the back and under the unharmed arm.
3. Use the clip to attach the free end of the neck strap to the brace.
4. Adjust to comfortable position. Unattach the loop of the free end and raise it to a comfortable position.

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Cautions and Warnings:

1. Read all the use instructions and warnings before use .
2. Consult your medical professional immediately if you feel any adverse reactions while using this product.
3. This product will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also essential parts of a safe treatment program.

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