

Waterproof Mattress Cover



Κάλυμμα Στρώματος Αδιάβροχο Υφασμάτινο

ref/κωδ. **0807863**

Single / Μονό



Dimensions / Διαστάσεις: 100x200cm

ref/κωδ. **0807864**

Double / Διπλό

Dimensions / Διαστάσεις: 200x200cm



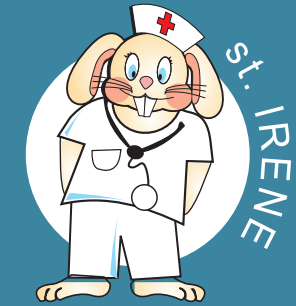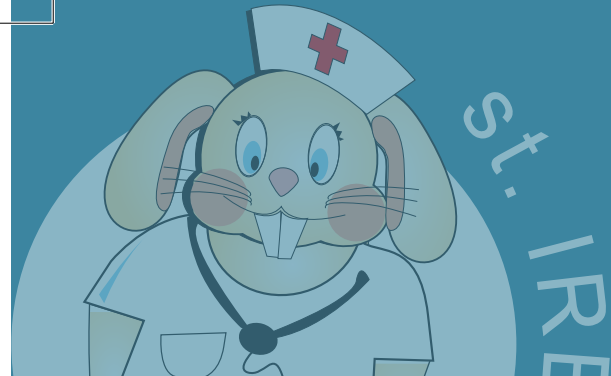

ΙΑΤΡΙΚΟΣ ΕΞΟΠΛΙΣΜΟΣ - HOME CARE



MEDICAL EQUIPMENT

EN

The **waterproof** mattress cover, protects the bed and maintains it clean and healthy from stains and dirt.

Prevents inflow of liquid.

It is easily placed around the angles of the mattress with the elastic loops and covers the whole surface.

EN

Placed onto the mattress and under the sheet.

Wash at 40°C.

Caution, wash the sheet with similar colors.

Avoid the wringing in the first wash.

Do not use detergents that include chlorine or bleaches.

Do not use Iron.

MOBIAK S.A.

96-98str. Markou Botsari

Chania - Crete

+30 28210 63222

www.mobiakcare.gr

GR

Το **αδιάβροχο** προστατευτικό κάλυμμα προστατεύει το στρώμα σας και το διατηρεί καθαρό και υγιεινό από λεκέδες και ακαθαρσίες.

Αποτρέπει την εισροή υγρών, όπως νερό και ούρα.

Τοποθετείται εύκολα στις γωνίες του στρώματος χάρη στα ειδικά λάστιχα, καλύπτοντας όλη την επιφάνειά του.

GR

Τοποθετήστε πάνω από το στρώμα και κάτω από το σεντόνι.

Πλένεται στους 40°C.

Προσοχή να πλένεται με όμοια χρώματα.

Αποφύγετε το στύψιμο στο πρώτο πλύσιμο.

Μην χρησιμοποιείτε απορρυπαντικά που περιέχουν χλώριο ή λευκαντικές ουσίες.

Το κάλυμμα δεν σιδερώνεται.

MOBIAK A.E.

Μάρκου Μπότσαρη 96-98

Χανιά - Κρήτη

28210 63222

www.mobiakcare.gr