EL - EN

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Function Anti - Trendelenburg



Libra / Virgo*

Fully electric wooden bed with adjustable height.

- Bed of steel frame in 4 sections
- Beech wooden cover
- Aluminum side rails
- 4 castors with brake
- 5 functions control motor with lock setting
- Included lifting pole

TECHNICAL DETAILS





Complete Bed dimensions	2160x1020 mm
Mattress Platform dimensions	2000x900 mm
Min. max. height	380x765 mm
Max inclination Backrest section	70°
Max inclination leg section	30°
Max inclination overall bed	20°
Max patient weight	170 Kg
Net weight	120 Kg
Laminate solid beech with vertical scroll system 80	370 mm
Μοτέρ - motor	24V
Τάση - voltage	220V50HZ

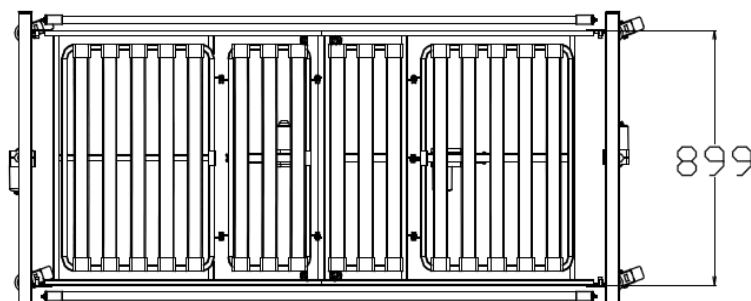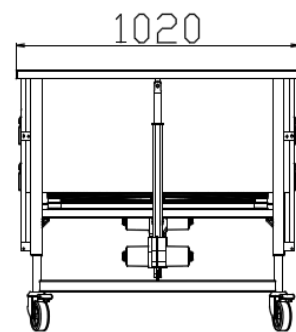TECHNICAL DETAILS 2 PRODUCT SCHEMATIC

Assembly

1. Open the box and take out of all the accessories from the box.
2. Place all the bed parts on the floor.
3. Assemble the bed surface and secure them with the 4 knobs which are in the box.
4. Assemble the headrest and footrest and secure them with the 4 knobs which are in the box too.
5. Assemble the headboard and lock the 4 knobs (knobs from parts box).
6. Install the piston type motor under the bed, but first take off the covers.
7. When you install the motor secure it with the covers you took off on the previous step.
8. Place the side rails and then attach the aluminum rails on the headrest and footrest..
9. Secure the side rails with the knobs which are in the box.
10. Plug in power.
11. Release the magnetic lock of the head controller.

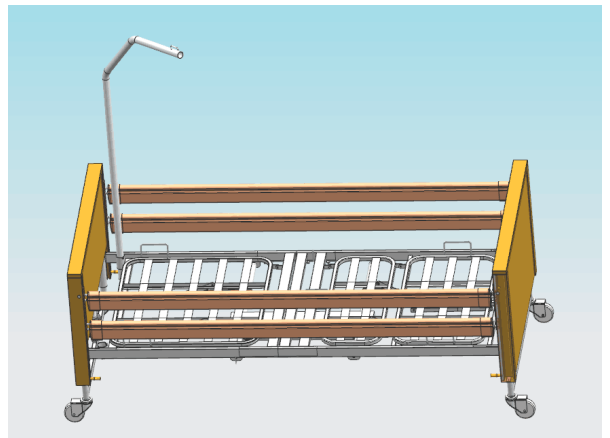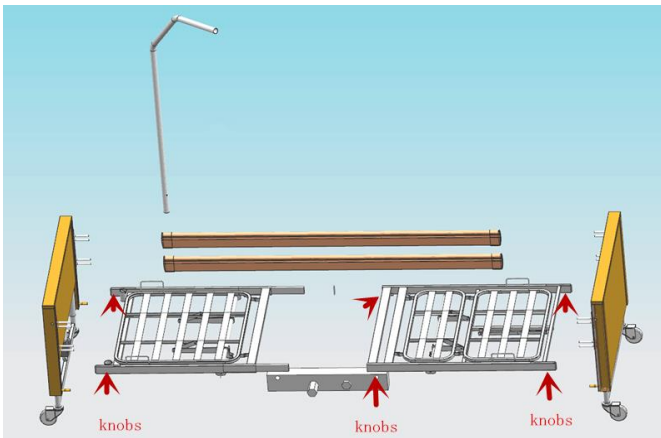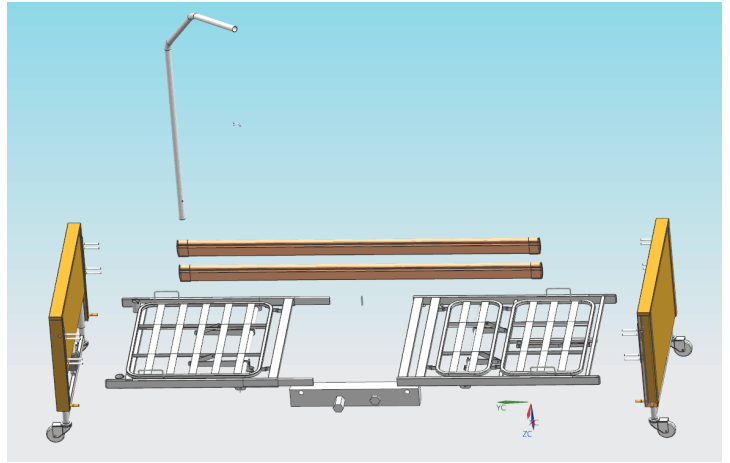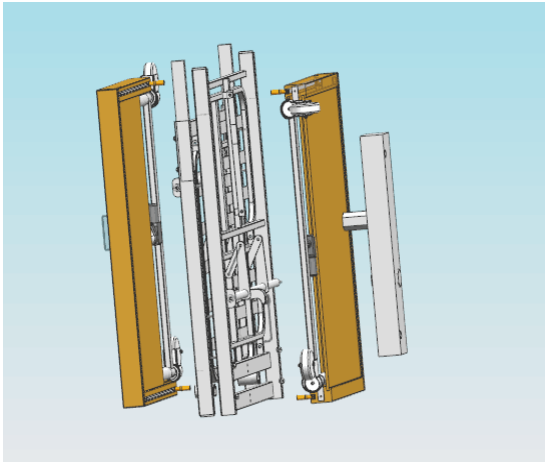PRECAUTIONARY INSTRUCTIONS

- (1) Maintain periodically.
Make sure all screws are secure and the cable has not deteriorated or frayed.
Make sure there are no cracks in the wood structure.
- (2) Do not exceed the recommended weight limit.
This could result in serious injury to the user.
- (3) Do not use chemical etchant cleaners to clean the frame. This could cause damage to the surface.
Use cold water and wipe with a dry cloth.
- (4) Make sure when you use home care electric bed, all four legs are on a solid surface and on the same level.

CLEANING

Use only a wet cloth and mild soap on surfaces where they need cleaning.
Then dry thoroughly before you use the product.

Never use acids, alkalis or solvents such as acetone or dissolving fluids.

IMPORTANT (!)

In case of failure or malfunction of the product your device, please contact your authorized and trained dealer.

Use of the following product is prohibited for purposes other than those specified in this manual

MOBIAS SA disclaims any responsibility for damage resulting from improper use of the product or use other than that set forth in this manual.

MOBIAS SA reserves the right to make changes to the product and the following manual without notice in order to improve its features.

EC DECLARATION OF CONFORMITY

We declare the conformity of the above product to the provisions of the Medical Equipment Directive 93/42/EEC and the Greek National Legislation ΔΥ8δ/Γ.Π.οκ.130648. ΦΕΚ 2198Β/02-10-2009

Each product in this statement meets the technical specifications contained in its technical documentation. If the appropriate maintenance will be provided and the product will be used according to the "instruction of use" then the product is safe and effective and does not endanger the health safety of patient's, user and or third parties,

This declaration is issued on the basis of a certificate of registration in the Manufacturer Register of Medical Devices of the Competent Authority (EOF).

The product belongs in Class I, Classification rule 1 & Corresponds to the Medical Device Directive: 93/42/EEC

Environmental Protection

If your product needs replacement, or you do not use it, consider the environment protection:

- 1) Do not dispose your appliance with other waste (this is the meaning of the recycled symbol).
- 2) Contact your local authority to instruct you the recycling centers for your appliance.
- 3) Recycling your no use appliance, you protect the environment and help the exploitation of new materials from your product.